

Jescope J-TD



Instruction for use :

Jescope J-TD is a tongue depressor to facilitate the insertion of LMA (Laryngeal Mask Airway).

Put the tongue depressor into the patient mouth, and then the slap bracelet will roll up patient's tongue.

Slide the LMA along the J-TD and place it deep into the patients' hypopharynx.

Cautions :

Do not expose to the sun or rain, keep the packaging intact, dry, and out of pollute.

Non-sterilized, single-used.

Do not use if allergy to silicon.

Lubricate J-TD and LMA with water-soluble jelly before use.

Materials : Plastic handle; steel core with silicone coverage.

